Kinder-Rückhaltesystem KIDDY · Art. 5800  
Children restraining system KIDDY · Art. 5800  
Basic-UDI-DI: 42506036RESQRUECK225W3

**RESCUEFORM®**

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### Intended use:

The children restraining system **KIDDY** is used only on stretcher in a Patient Transport Ambulance for the transportation of injured children with an **age from 7 month (ca. 80 cm) until 7 years (ca. 130 cm) with a maximum weight of 25 kg**, if their status requires an immobilization. After the child was put on the vacuum mattress, the mattress can be moulded to the body and be fixed through a vacuum afterwards.

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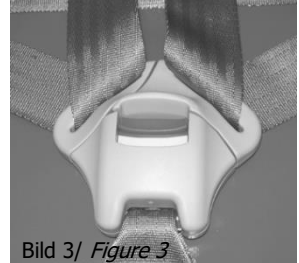
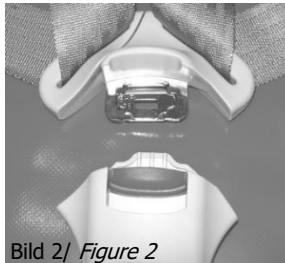
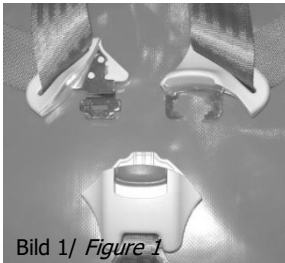
**Safety blanket belts**

To fortify the security-cover on the stretcher/trolley, fix the belts in the head- and foot-area and on both sides at the beams of the stretcher/trolley. Ask the manufacturer of driving-appliance and stretcher/trolley for the best suitable position. Be sure that all belt-buckles are properly engaged. The belts must be tightened in order to prevent a slipping of the security-cover.

**In order to guarantee maximum security, an eye must be kept on that the blue belt of the foot-area is not fixed on the foot-bar of the stretcher.**

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*The patient belts are used to hold the patient on the safety blanket. The lock continues 3 parts (FIG. 1). The patient belts on top of the left and the right side are going through the two small lock parts (L = left side of patient / R = right side of patient). They are separated in shoulder- and waist belt. The main lock (M) is connected with the gusset strap.*

*To close the lock please put the fit Part (L) onto the fit part (R) (Fig. 2) and fix them together into the main lock (M) (Fig. 3). When fitting the belts into the 3-point-lock be sure that they lock audibly into place. Now fasten the side belts on the left and the right side.*

**Recommendation: Practice fitting the belts to the lock several times before initial use, checking every time that the clips are properly fastened.**

Use the belts of the stretcher/trolley additionally for the patient-fixation over hip and legs.

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**Vacuum mattress with cover**

*The vacuum mattress can easily be fitted over the safety blanket if required. To do this, first attach the safety blanket to the stretcher/trolley and open the lock of the patient belts. Then place the vacuum mattress on top of it (head-area in the direction of travel). Then place the patient on the vacuum mattress and mould the content. Pump off the inner-air and secure the child with the patient belts. The Velcro and fleece belts attached to the top of the vacuum mattress can be used as head restraint. The blue cover can be decreased.*

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**Security informations**

*During handling of vacuum-rescue-articles the following points must be heeded for the avoidance of damages particularly during the use urgently:*

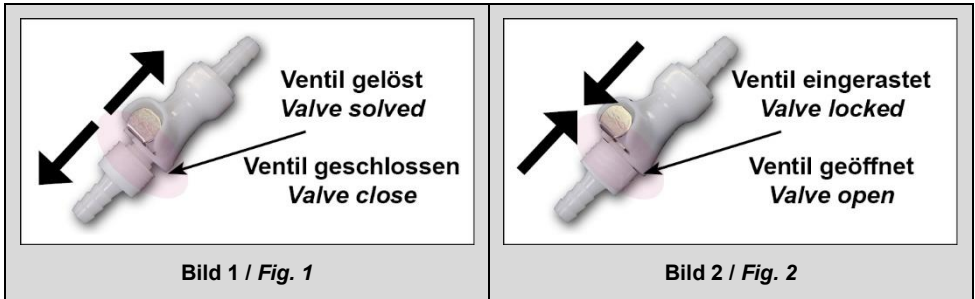
**Don't hurt the foil with pointed objects (e.g. cannulas or scissors)! When putting down the vacuum-articles always ensure that the surface is clean (e.g. no broken glass, sharp pebbles etc.). A damaged and leaky vacuum mattress can't sufficiently stabilized lasting through a vacuum. It loses the fixing effect within short time!**

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**Valve for one-hand-operation**

**VACUFORM** is equipped with a one-piece valve that can easily be operated with one hand due to its small size. It is permeable to x-ray and MRI. In order to open the valve, it must be telescoped until it snaps in audibly (**Fig. 1, page 4**). It is closed by pushing the silver button. The telescoped valve pops apart a bit and the airflow shuts down (**Fig. 2, page 4**).

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### Repair

*Repair sets including instructions can be obtained from us for repairing minor damage to the vacuum mattress (needle pricks, etc.). No special instruction is required. Repairs carried out must be checked again. Further use may only take place after passing the test. If more extensive repairs are required, these may only be carried out by the manufacturer.*

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### Liability for material defects:

*We provide free rework or free replacement for material or manufacturing defects (material defects) within 12 months from the date of manufacture. This excludes damage caused by mechanical (e.g. needle sticks in the vacuum mattress) or chemical (e.g. harsh disinfectant) influences during use or storage.*

**Vacuum mattress:** *With a frequent use and/or a commonly evacuation (vacuum), the volume of the filling-material can decrease in the course of the time. This is a normal abrasion, which is not implied by the guarantee.*

*We are not liable for direct or indirect damage of any kind to persons or objects resulting from improper use, disregard of the instructions for use or the impossibility of use due to neglected or incomplete repairs. (Medical Device Operator Ordinance)*

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**Tasks and duties for the operation of a medical device**

*The operator of a medical device is responsible for the commissioning and operation of the product. This means that the legal provisions must be observed and implemented. The operator must create the organizational requirements for the intended use of the product and ensure that the operating personnel have the necessary expertise and reliability. The operator's duties also include ensuring the functionality and proper condition of the product.*

*Maintenance may only be entrusted to persons who have the necessary expertise. The exact wording of the requirements of all responsibilities of the operator of medical devices can be found in the current legal provisions of the country in which the product is used.*

*For disposal or recycling of the product, please contact your local waste disposal company. If the product is returned free of charge, we will add it to the raw material cycle in an environmentally friendly manner.*

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**The safety blanket including belts must be subjected to a regular, at least annual, inspection by an expert. If any damage is found, the safety blanket must be withdrawn from further use immediately. A repair of the safety blanket is not possible for safety reasons.**

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**Parts of the KIDDY**

- Art.-No. **5801** KIDDY safety blanket, yellow, with blue belts  
 Art.-No. **5802** KIDDY vacuum-mattress, light blue, size 40 x 100 cm  
 Art.-No. **5803** KIDDY cover for the vacuum-mattress, blue  
 Art.-No. **5804** KIDDY bag

All parts are individually available as replacement part.

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**Cleaning / disinfection**

<b>Warning notice:</b>	<i>The surfaces must not be treated with corrosive cleaning agents or disinfectants. Damage to the surface may lead to leakage, but in any case to reduced flexibility of the film. The product loses its intended properties and becomes unusable.</i>
<b>Restrictions:</b>	<i>The products must not be cleaned and disinfected thermally or by machine.</i>
<b>First handling:</b>	<i>Application-specific (Germany: MPBetriebV), in accordance with the relevant authorities.</i>
<b>Cleaning preparation:</b>	<i>Ventilate the vacuum article and spread it out without wrinkles.</i>
<b>Machine cleaning:</b>	<i>The products are <b>NOT</b> suitable for machine cleaning.</i>
<b>Manual cleaning:</b>	<p><i>It is recommended to clean / reprocess vacuum film items immediately after they have been used, as blood and colored disinfectants are extremely aggressive. Discoloration can no longer be completely removed after a longer exposure time.</i></p> <p><i>Equipment: clear water, mild soapy water, soft brush or sponge, disposable cloths (according to requirements).</i></p> <ol style="list-style-type: none"> <li><i>1. Remove surface contamination with clear water and a soft brush, sponge or lint-free cloth. Treat all surfaces. Rinse with clean water.</i></li> <li><i>2. Apply mild soapy water to all surfaces with a soft brush, sponge or lint-free cloth, wipe and rinse with clear water. Clean the belt material with the brush and rinse it too.</i></li> </ol> <p><i>Pay attention to areas that are difficult to access!</i></p>
<b>Disinfection:</b>	<p><i>Non-critical products. A low level of disinfection must be carried out.</i></p> <p><i>Use a commercially available wipe disinfectant solution for sensitive plastic surfaces in accordance with the manufacturer's instructions for use. Use a maximum of 2% solution. Pay attention to areas that are difficult to access!</i></p> <p><i>Spray existing uncoated / textile belt material (restraint systems) with the disinfectant solution, moisten sufficiently. Observe the manufacturer's information on the exposure time!</i></p> <p><i>A book must be kept of the disinfection carried out (disinfection protocol)</i></p>
<b>Drying:</b>	<p><i><u>After cleaning:</u> wipe with dry disposable cloths, air dry / allow residual moisture to evaporate.</i></p> <p><i><u>After disinfection:</u> do not rub dry! Adhere to the exposure time for the disinfectant specified by the manufacturer! Allow residual moisture to dry off / evaporate. Wipe with a clean cloth (e.g. disposable cloth) only if necessary (emergency).</i></p>
<b>Maintenance:</b>	<i>The products do not have any components that are subject to usage-related wear. Maintenance is therefore not required.</i>
<b>Control / functional test:</b>	<p><i>Regular inspections in accordance with the respective competent authority (Germany: MPBetriebV).</i></p> <p><i>Visual inspection including existing belts, check vacuum items for leaks (test duration at least 120 min.), Functional test of existing belt buckles and the valve.</i></p>
<b>Packaging:</b>	<i>If necessary, storage in plastic bags of the appropriate size or in a transport bag made of nylon fabric (available as an accessory).</i>
<b>Sterilization:</b>	<i>Not required as it is not used in areas that are to be kept sterile.</i>
<b>Storage:</b>	<p><i>After complete drying at room temperature, avoid direct sunlight, if possible without a vacuum.</i></p> <p><i>If the spatial conditions require it, the product can be stabilized by a slight vacuum. Please follow the instructions for use!</i></p>
<b>Additional information:</b>	<i>Not mandatory.</i>