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**10-PUNKT FIXIERSYSTEM
10-POINT RESTRAINT SYSTEM**

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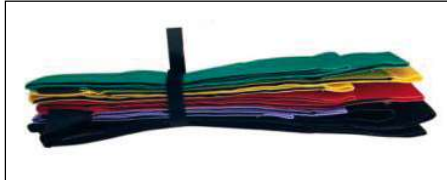
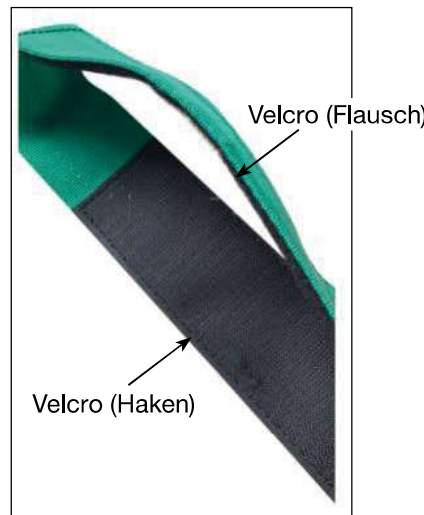
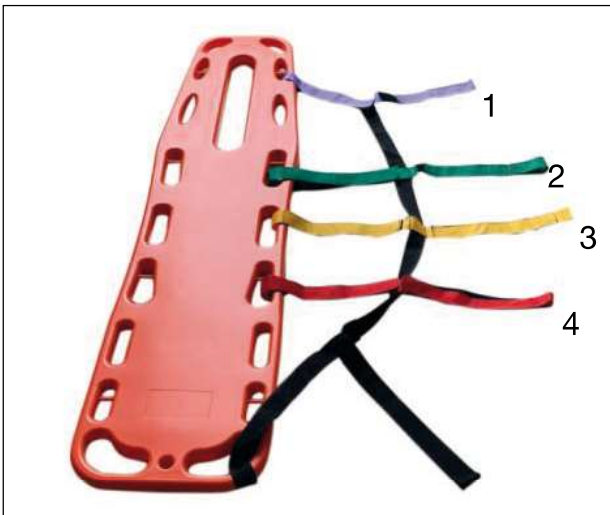
LIFEGUARD 10-POINT RESTRAINT SYSTEM FOR SPINEBOARD

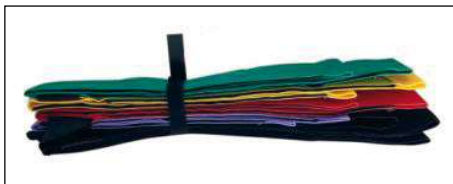
adjustable, according to PHTLS guideline, incl. bag.

I. INTENDED PURPOSE


The belt system is used to secure a wounded person/patient during transport on a spineboard.

II. USE METHOD

1. Take out the spider strap from the packaging bag, loosen the velcro on the safety belt, and lay the safety belt flat.

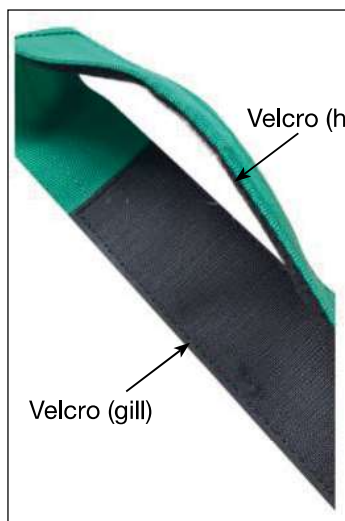2. Put the wounded or patient on the stretcher, and fix the safety belt in each hole of the stretcher according to the height of the patient body (by moving the belts 1-4 to adjust the corresponding position, each belt has a velcro, through the velcro fix it in position).

3. Cover the wounded or patient with the belt, and fix the belt at the other end through the hole (according to the body of the wounded or patient, use the velcro on the belt to protect it from looseness or tightness). Then transfer the patient.

III. SAFETY INSTRUCTIONS

1. Please check whether the belt is worn or not before use. If it is worn, please replace it in time.
 2. Before transferring the wounded or patient, make sure that the belts are secured to prevent the belts from loosening and causing personal injury.
- ⚠ All serious incidents related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient is established.

Erstellt am:/Created on: 2024-10-29

 JIANGSU RIXIN MEDICAL EQUIPMENT CO., LTD. No. 427 Yangjin Road, Jinfeng, Zhangjiagang, Jiangsu Province, China

 SUNGO Europe B.V., Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel, The Netherlands

 servoprax GmbH, Am Marienbusch 9, 46485 Wesel, Germany
www.servoprax.de, info@servoprax.de

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1-N5 ST02035Aff-075-2-0004-2411