REF N6 SH00111C  
REF N6 SH00121B

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## en | INSTRUCTIONS FOR USE

### INTENDED PURPOSE

The head immobilizer is intended for securing or supporting injuries to the cervical spine in first aid or physiotherapy. It is used to immobilize the patient's head.

### INDICATION

1. For fixation of the head during first aid.
2. For preoperative and postoperative stabilization of the cervical spine.
3. For immobilization in case of cervical spine injuries.
4. For retrograde injury of the cervical spine.
5. For ligament or soft tissue injuries in the cervical region.

### DIRECTIONS FOR USE

Instructions for the user:

1. Attach the head support to the base plate.
2. Place the patient on the scoop stretcher and attach the two head pads to the right and left of the patient's head.
3. Secure the patient's head with two forehead straps.

### NOTES

1. The special head and chin straps are designed to fit the patient's forehead.
2. Large ear holes to monitor the patient's ear canal.
3. Minimal interference with X-ray, MRI or CT scan.
4. Soft foam with waterproof vinyl cover, easy to clean.
5. Compatible with all scoop stretchers, spineboards and combicarriers.

### CLEANING AND CARE

1. Do not use aggressive substances. Clean with water and a gentle detergent or mild soap.
2. Dry the head fixation with a soft cloth after cleaning and make sure that no detergent residue remains.
3. For disinfection, use only disinfectants that are specifically approved by the manufacturer for PE foam and vinyl/plastic.
4. The head fixation must not be cleaned or sterilized by machine.

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### STORAGE

Avoid high temperatures and humidity.

### SHELF LIFE

Five years

### PRECAUTIONS

1. Improper use of the head fixation may result in injury. Use the head fixation only for the purpose described in the instructions for use. Do not use the accessory for any purpose other than its intended use.
2. Untrained personnel may cause injuries or be injured themselves. Only trained personnel are allowed to use the head fixation.
3. An unattended patient can be injured. Stay with the patient at all times.
4. Improper maintenance can cause injuries. Maintain the head fixation only as described in these instructions.
5. Improper parts and maintenance can lead to injuries. Use only original accessories and have maintenance performed only by qualified personnel.
6. Do not use the product if it is severely deformed or damaged.

### CONTRAINDICATIONS

None.

#### ⚠ Warning

Please use the head fixation only as trained personnel.

⚠ All serious incidents related to the product must be reported to the manufacturer and the competent authority of the Member State where the user and/or patient is established.

### DISPOSAL

Please dispose of the product at the end of its shelf life in accordance with local regulations.

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