Geratherm[®]
flex

Geratherm[®]
Baby flex

GT-3020

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Instruction Manual (EN)

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Bedieningshandleiding (NL)

CE0197

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
PRODUCT DESCRIPTION

Package contents

- 1 x Medical thermometer
- 1 x Protective case
- 1 x Instructions for use
- 1 x 1,5V button cell LR41

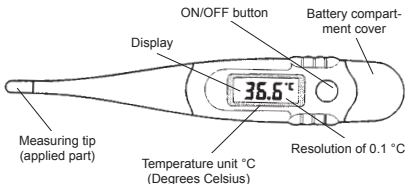MEASURING PRINCIPLE

A temperature-dependent sensor is located in the measuring tip of the thermometer. Over the duration of the measurement, the temperature of the measuring tip adapts to the temperature of the surrounding tissue. Electronics integrated in the thermometer convert the temperature recorded by the sensor into a digital value and show this as a measured value on the display.

MEASUREMENT TYPES

There are different temperatures at each measuring point of the body according to the balance between heat generation, transfer and loss. **We recommend rectal or oral measurement.**

Rectal measurement - temperature measurement in the anus

Lubricate the measuring tip with suitable lubricant for an easier insertion. Then carefully insert the measuring tip into the anus (rectum) between 2 to 3 cm.

Oral measurement - temperature measurement in mouth

Place the measuring tip into one of the two heat pockets under the tongue to the left or right of the root of the tongue. Close your mouth and breathe evenly through the nose.

Axillary measurement - temperature measurement under the armpit

Dry the armpit with a towel and place the measuring tip in the armpit. Close the arm tightly.

WARRANTY

This thermometer has a two-year warranty against defects caused by the manufacturer from the date of purchase. Please refer to the notes and information in our guideline on dealing with claims.

INTENDED PURPOSE

The medical thermometers Geratherm® flex and Geratherm® Baby flex are intended for rectal, oral or axillary body temperature measurement in home and professional healthcare environments to be operated by adults only.

Indications

- Suspected fever or elevated body temperature (hyperthermia)
- Suspected decrease in body temperature (hypothermia)
- Monitoring for disease progression with fever (temperature monitoring)

Contraindications

The Geratherm® flex and Geratherm® Baby flex must not be used under the following conditions:

- By users and patients who are mentally or physically incapable of using the thermometer in accordance with these instructions for use.
- This thermometer is not intended for measuring basal body temperature.

Patient target group

Within the specified measurement range and measuring types, the device can be used for all patients irrespective of age, bodyweight or gender.

Intended user

The device may only be operated by adults or under supervision of an adult. All persons handling the product should have read the instructions for use carefully.

PRECAUTIONS AND WARNINGS

- ⚠ **Failure to comply with the intended use and a lack of knowledge on use of the device can lead to incorrect measurement results and misdiagnosis.**
 - Read the instructions for use completely before using the thermometer for the first time and observe all recommendations for use as well as the warnings and precautionary measures contained in the instructions for use.
 - Only use the product in accordance with these instructions for use.
 - Consider all information on the product and its packaging.
 - Only use the product at the intended measuring locations.
 - Keep the instructions for use for additional users and future applications.
- ⚠ **Using the thermometer is not a substitute for consulting a doctor.**
 - Consult your doctor if the fever persists or has a high temperature.
- ⚠ **Modifications to the thermometer can lead to injuries to the patient or to the loss of function of the thermometer.**
 - Do not modify the device.
 - Only use the device when it is intact.
- ⚠ **Inaccurate temperature measurements can lead to misdiagnoses and incorrect therapies.**
 - Axillary measurement potentially leads to inaccurate readings.
 - More precise measurement results are obtained with rectal and oral measurements.
 - In the case of critical measured values, a rectal or oral control measurement should be performed.
- ⚠ **The thermometer used and the measurement location have an influence on the measurement result.**
 - Only compare measured values that were determined at the same measuring location and with the same thermometer.
- ⚠ **Incorrect procedure of temperature measurements can lead to false readings and a misdiagnosis.**
 - Do not take a temperature reading within 30 minutes of exercising, bathing, eating, drinking or smoking.
 - Make sure that the measuring tip is placed in one of the two heat pockets under the tongue, to the left or right of the root of the tongue when taking oral temperature.

- Keep the mouth closed during oral temperature measurement and continue breathing evenly through the nose to establish thermal equilibrium.
- Keep the armpit closed during the axillary temperature measurement to establish thermal equilibrium.
- Wait for the end of the temperature measurement, signalled by a series of beeps and the end of the flashing °C symbol, before removing the thermometer.

⚠ Excessive force when inserting the thermometer may cause injury to surrounding tissue, e.g., perforation, puncturing.

- Do not force insertion and stop if you feel resistance.

⚠ This thermometer contains small parts that can be swallowed and pose a choking hazard for children.

- Keep the thermometer out of the reach of children.
- Temperature measurement in children must only be performed by an adult.

⚠ The possibility of allergic reactions and / or skin irritations cannot be completely ruled out.

- If allergic reactions or skin irritation occur, stop the measurement and avoid using this thermometer again.

⚠ Germs, e. g. bacteria and viruses, can be transmitted from measuring site to measuring site and from patient to patient during use and cause infection.

- Clean and disinfect the thermometer before first use and after each measurement.
- If contamination or liquids visibly penetrates the housing, the thermometer can no longer be reprocessed hygienically and must be disposed of.

⚠ Incorrect storage or transport can damage the thermometer.

- Always keep the thermometer away from direct sunlight, temperatures above 60 °C, moisture and dirt.

⚠ Incorrect handling can damage the thermometer.

- Avoid contact of the thermometer with temperatures higher than 60 °C.
- Always keep the thermometer away from direct sunlight.
- Always keep the thermometer away from moisture and dirt.
- Do not excessively bend the tip of the medical thermometer.
- Do not bite the thermometer during use.

⚠ Defective thermometers lead to incorrect measurement results.

- Protect the thermometer from knocks and bumps.
- Always store the thermometer in the protective cover provided.
- Before each use, check your thermometer for functionality, external damage and signs of excessive wear.
- Verify the result of the self-test on start-up.
- Check the measuring accuracy at least every 2 years as part of a metrological check.
- Check the measured value obtained by means of a control measurement if you are unsure or the measured value deviates greatly from the expected measured value.
- Do not use the thermometer if it is damaged or the measuring accuracy is no longer given.

⚠ Batteries can leak and damage the product.

- Remove the batteries if the thermometer is not going to be used for a long period.

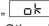
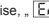
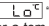
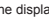
⚠ Improper disposal can lead to environmental damage.

- Dispose of used batteries according to local regulations.
- Dispose of the thermometer in accordance with local regulations for electronic waste.

APPLICATION

TEMPERATURE MEASUREMENT

How to operate

1. Remove the thermometer from the protective case.
2. Before using the thermometer for the first time, clean and disinfect it according to the processing instructions in Chapter "Processing".
3. Press the "ON/OFF button" to switch on the thermometer.
4. A full display „“ ensures that the display is fully operational.
5. The next appearance „“ signals normal operation after internal safety testing. Otherwise, „“ shows a malfunction.
6. Then, the display shows the ambient temperature or „“ while °C is blinking. The thermometer is now ready for a temperature measurement. If the thermometer is not used within 3 minutes, the thermometer will be automatically turn-off.
7. Put the probe head in the rectum, under the tongue or under the arm according to the methods provided. We recommend determining the body temperature using the rectal or oral measurement method.
8. When the thermometer completes its temperature measurement, °C stops blinking and the measured temperature displays with a series of beeps.
9. When the measured temperature is above or below the measuring range, the display shows „“ or „“.
10. Please turn off the thermometer when not in use. An automatic switch-off occurs after 9 minutes.
11. Prepare the thermometer for storage and for the next use according to the information in chapter "Processing".

NOTE! If the thermometer has been stored outside the intended working temperature of +5 °C to +40 °C, or if the ambient temperature is higher than the expected reading, immerse the probe tip in cold water for 5-10 seconds before use.

Evaluation of the measured values

The values listed below describe an average normal body temperature, you can use them as a guide:

- For rectal measurement (in the anus):
34.4 - 37.8 °C
- Oral measurement (in the mouth):
36.0 - 37.6 °C
- For axillary measurement (armpit):
35.5 - 37.0 °C

Please note, however, that the body temperature depends on the time of day and the degree of activity, as well as on physiological factors such as age, etc., and that the clinical literature also describes values that deviate from those mentioned.

An interpretation of the measured value must always be made on a case-by-case basis.

In the case of a measured value above the normal temperature, there is an increased body temperature. Fever (hyperthermia) can be assumed.

A reading of 39 °C or higher, regardless of the measurement site, indicates high fever. For infants, a paediatrician should be consulted if the reading is 38.0 °C or higher.

In the case of a measured value below the normal temperature, there is a reduced body temperature. Hypothermia can be assumed.

CALLING UP MEMORY VALUES

How to recall the last temperature?

1. Press the ON/OFF button down for approximately 1.5 seconds.
2. The last temperature will be recalled on display with "M" flag appearing at the bottom-right corner.


For example: 

CHANGING THE BATTERY

When the battery flag "" appears and blinks, it means time to replace the battery.

In this case:

Note: it is important to turn the thermometer off before replacing the battery, otherwise the thermometer may malfunction.

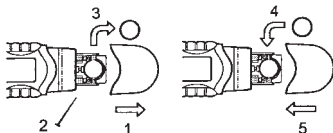1. Release the battery compartment cover by gently pressing on the catch.
2. If necessary, use a pointed object, but not metal to remove the old button cell.
3. Remove the old battery.
4. Replace with a new button cell 1.5 V of the same type (LR41), with the cathode side (-) upward.
5. Close the battery compartment cover.

Note: dispose of used battery promptly according to the instructions given in chapter "Disposal".

PROCESSING

The instructions provided below have been validated by the manufacturer of the medical device as being capable of preparing the thermometer for re-use. It remains the responsibility of the processor to ensure that cleaning and disinfection, as actually performed, achieves the desired result.

⚠ Any processing other than that described in this document can damage the thermometer.

- Do not clean the device in an ultrasonic cleaner.
 - Avoid automated cleaning and disinfection in a washer-disinfector (WD).
 - Do not sterilize the thermometer.
 - Do not expose the thermometer to temperatures greater than 60 °C.
 - Do not immerse the thermometer in liquids and any cleaning/disinfection solution.
 - Avoid excessive moisture during processing.
- ⚠ The use of chemicals can damage the thermometer or harm the user.**
- Please note the instructions and warnings for the cleaning and disinfectant agent of the respective manufacturer.
 - If necessary, wear disposable gloves and protective goggles during processing.
 - Do not use strongly acidic or alkaline cleaning agents.
 - Do not use cleaning agents based on quaternary ammonium compounds.
 - Do not use solvents (e.g. acetone, benzene etc.).
 - Do not use strong oxidants (e.g. bleach, peroxides etc.).

Manual cleaning

- Make sure that all surfaces of the thermometer can be reached during the cleaning process.
- Remove coarse soiling and lubricant residues in advance of processing with a dry and clean disposable cloth.
- To clean the thermometer, first prepare a cleaning solution by mixing 1 ml of a pH-neutral dishwashing detergent (Pril Original has been validated) in 1 L of water (at least drinking water quality) at a temperature of approx. 35 °C.
- Moist a clean soft cloth with this cleaning solution and wipe the entire surface of the thermometer thoroughly towards the tip.
- Repeat the wipe cleaning two to three times.
- To remove cleaning agent residues, moist a new, clean, soft cloth with pure water (at least drinking water quality) and wipe the thermometer several times.
- Any remaining moisture can be removed with a dry and clean cloth. If soiling is still visible after cleaning, repeat the above steps.

Manual disinfection

- Make sure that all surfaces of the thermometer can be reached during the disinfection process.
- To disinfect the thermometer, moist a clean, soft cloth with an alcohol-based disinfectant agent.
- The effective disinfection of the thermometer has been validated with 70 % isopropanol with an exposure time of 5 minutes.
- Follow to the instructions for use of the disinfectant agent manufacturer.
- Wipe the entire surface two to three times.
- Any disinfectant residues remaining after the exposure time can be removed with a dry and clean cloth.

Drying

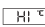
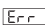
- Let the thermometer air-dry completely on a clean surface at ambient temperature (15 °C to 25 °C).

Storage

- Store at a dry, clean and dust-free place. Use the protective cover supplied with the thermometer.

SERVICE AND MAINTENANCE

Display Icons and Troubleshooting

Display	Description
No display	Please check the battery first before sending it for repair.
	Battery is empty.
	Measured value is above 43.9 °C and therefore outside the measuring range.
	Measured value is under 32.0 °C and therefore outside the measuring range.
	Electronic failure. If this occurs repeatedly, contact the manufacturer.

METROLOGICAL CONTROL

It is recommended to have a metrological check on the thermometer carried out by an authorized laboratory at least every 2 years. This ensures the correct function and measurement accuracy.

DISPOSAL

When disposing of the thermometer, batteries and packaging, the applicable country-specific regulations must be observed.

- Dispose of the packaging of your thermometer according to local regulations.
- Remove the battery before disposing of the thermometer.
- Do not throw used batteries in the household waste, but dispose of them as hazardous waste or at a battery collection point in specialist shops.
- Dispose of the thermometer in accordance with local regulations for electronic waste.
- Do not dispose of the battery or thermometer in a fire.

GUIDELINES AND MANUFACTURER'S DECLARATION REGARDING ELECTROMAGNETIC COMPATIBILITY (EMC)

⚠ WARNING:

Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

⚠ EM disturbances can affect the measurement accuracy of the thermometer.

⚠ WARNING:

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the GT-3020, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

The complete EMC appendices are available on request.

TECHNICAL DATA

Type of thermometer	Medical thermometer, direct mode
Mode of operation	intermittent
Transient time	60 seconds
Measuring range	32.0 °C - 43.9 °C
Measuring accuracy	± 0.1 °C (in the measuring range 34.0 °C – 43.0 °C) ± 0.2 °C (in the measuring range 32.0 to 33.9 °C and 43.1 to 43.9 °C)
Display resolution	0.1 °C
Unit	Degree Celsius (°C)
Power supply	1 button cell 1.5 V LR41 or comparable (4000 measurements)
Weight / dimensions	Appr. 17g 133.5 mm x 22 mm x 11.5 mm
Degree of protection	IP22
Working conditions	Temperature: +5 °C to +40 °C Humidity: 15 % to 90 % rel. Humidity Air pressure: 700 – 1060 hPa
Storage and transport conditions	Temperature: -25 °C to +60 °C Humidity: 15 % to 90 % rel. Humidity Air pressure: 700 – 1060 hPa
Life time	5 years
Additional functions	Self-test, memory, Beeper

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Geratherm Medical AG