

# Product Data Sheet

## Softtemp

### Disposable thermometers covers




## Thank you for choosing Softtemp covers!

### About company:


EQUA is a manufacturing company in Estonia, established in 1995. EQUA mission is to offer comprehensive solutions to the hospitals in the field of medical products and furniture.

### Product description

#### Inner cover:

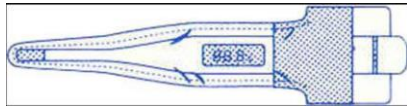

#### Outer cover:

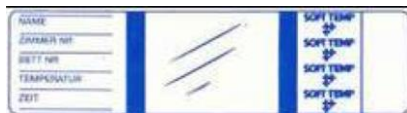

#### SOFT TEMP Digital

These disposable covers are specially designed for digital thermometers. PVC free! The lower section of the outer cover is non-transparent. Space provided for noting patient's name, bed number, room number, temperature and time.

#### SOFT TEMP Digital – lubricated

These disposable covers are specially designed for digital thermometers and for rectal use. Lubricated with glycerin ointment. The lower section is non-transparent. Space is provided for noting patient's name, bed number, room number, temperature and time. PVC free!

#### SOFT TEMP Standard

These disposable covers are specially designed for mercury thermometers. PVC free! The lower section is non-transparent. Space provided for noting patient's name, bed number, room number, temperature and time.

#### SOFT TEMP Standard – lubricated

These disposable covers are specially designed for glass/mercury thermometers and for rectal use. Lubricated with glycerin ointment. The lower section is non-transparent. Space provided for noting patient's name, bed number, room number, temperature and time. PVC free!

### How to use the SOFT TEMP?

**Step 1: Open the small carton and take out one cover.**

**Step 2: Insert the thermometer into the inner cover of the SOFTTEMP product.**

**Step 3: Hold the inner cover with one hand and the outer cover with the other and pull the inner cover out together with the thermometer.**

**Step 5: Use the thermometer for its purpose. If required write the notes about the patient on the outer cover – name, room number, bed number, temperature and time.**

**Step 6: Keep the thermometer inside the inner cover at all time while taking the temperature. Insert the inner cover together with the thermometer back into the outer cover after the temperature has been taken.**

**Step 7: Remove the thermometer from the product and dispose both covers.**

**Step 8: The thermometer is ready for use with the next SOFTTEMP cover.**

Data Sheet	English	German
Purpose & Application area	Non-sterile disposable product for covering thermometer to minimise the risk of gross-infection. For single use only.	
Directives /Certificates	Directive 93/42/EEC, class I Satute of Ministry of Social Affairs in Estonia no. 41 (RTL2000,6,997 §§ 22-75) Declaration of Conformity	
Directions of use	instructions for use are listed in user manual: · harmless, if used properly by qualified persons · only use undamaged products! Attention to the integrity of the article must be paid before and during usage! · the user is expected to prove the suitability of the product for its intended use at his own responsibility! · special attention must be given if used on babies and rectally due to fragile skin.	
Restrictions of use	not for activities with hazardous, sharp or aggressive substances! Only for single usage. In threat of contamination the product must be disposed before use! If there is a threat of harming the skin (in rectal area) or existing wounds the rectal thermometer taking should not be performed!	
General safety notes	· Do not use near strong heat or open fire! · Do not tear, stretch or damage the cover before use!	
Material	PVC and Latex free	
Inner cover material characteristics	PE film, EBA	
- characteristic	Heat welded seals, stretchable, partially transparent	
- thickness	0,035 mm (+/- 0,005 mm)	

- minimum self-life	3 years from production date if stored correctly	
Outer cover material characteristics	PE film	
- characteristic	Heat welded seals, not stretchable, transparent, printed area not transparent	
- thickness	0,04 mm (+/- 0,01 mm)	
- minimum self-life	3 years from production date if stored correctly	
<b>Paper</b>	70 g/m2, 20% PE coating, white	
Oitment characteristic	Glycerin 86,5 % vegetable	
Product types		
- softtemp digital	H7 1210	
- softtemp digital lubricated	H7 1208	
- softtemp standard	H7 1110	
- softtemp standard lubricated	H7 1108	

Producer info:

EQUA OÜ  
Lembitu 8, Tartu 50406  
Estonia  
Tel. +372 5659040  
[www.equa.ee](http://www.equa.ee)  
[info@equa.ee](mailto:info@equa.ee)