




Medline Industries Ltd  
3rd Floor, Quayside Wilderspool Business  
Park  
Greenalls Avenue  
Warrington WA4 6HL  
United Kingdom

Tel.: +44 844 334 5237  
Fax: +44 844 334 5238  
[www.medline.eu/uk](http://www.medline.eu/uk)

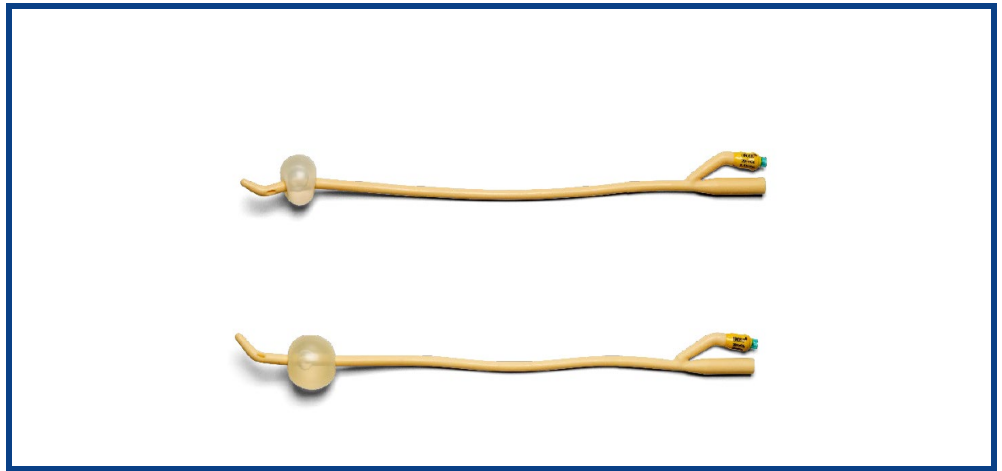

## Technical Sheet

# UROSID<sup>®</sup> Latex Foley Catheter

Transurethral Latex Foley Catheter, Tiemann, double lumen, sterile

### Reference(s)

Reference	Size (CH)	Length (cm)	Balloon volume (ml)	Color	Pack factor
665012A	12	40	5-10	White	10
665014A	14	40	5-10	Green	10
665016A	16	40	5-10	Orange	10
665018A	18	40	5-10	Red	10
665020A	20	40	5-10	Yellow	10
666012A	12	40	30	White	10
666014A	14	40	30	Green	10
666016A	16	40	30	Orange	10
666018A	18	40	30	Red	10
666020A	20	40	30	Yellow	10

### Description

The UROSID<sup>®</sup> Latex Foley Catheter is a double lumen, transurethral bladder catheter made of siliconized Latex. It is designed with a Tiemann tip matching especially the needs of male patients with and enlarged prostate or narrowed urethra.

UROSID Foley Catheters are color-coded for easy selection of the correct CH-size. UROSID Latex Foley catheters are equipped with a balloon to secure the catheter inside the patients' bladder.

The balloon has a capacity of 5-10ml or 30ml, depending on the item code. The 30ml balloon is larger than the average balloon size to fill additional cavity inside the bladder. For blocking of the balloon, a blocking medium is necessary. We recommend Aqua-SID (100% sterile water) for all UROSID Latex Foley catheter. Only block the balloon with the maximum capacity mentioned on the blocking funnel using a Luer syringe.

Indwelling time: up to 14 days.

#### Not to be used in cases of:

Injuries of the urethra or stenosis, acute inflammation of the urinary tract (urethritis, prostatitis, epididymitis), or in case of known hypersensitivity to the materials used.

### Main materials

Component	Materials
Catheter	Natural rubber latex, siliconized
Balloon	Natural rubber latex
Catheter tip	Natural rubber latex, siliconized
Check valve	PVC, PP, ABS, Silicone, Stainless Steel



<b>Pack factor</b>	Refer to <b>Reference(s)</b> section
<b>Packaging</b>	Double sterile peel pouch
<b>Legal Manufacturer</b>	Well Lead Medical Co., Ltd., C-4 Jinhua Industrial Estate, Hualong, 511434 Panyu, Guangzhou People's Republic of China In compliance with EN ISO 13485
<b>Regulation</b>	In compliance with Medical Devices Regulation 2017/745 and UK MDR MD Class: IIb sterile  •Please refer to complete labeling before using the product.
<b>Other Information</b>	Sterile Ethylene oxide (EO) Shelf Life 5 years Storage - In a dry place, away from sunlight

