



Rhino  
Rescue


# CHEST SEAL

-ALWAYS BE PREPARED-

# CHEST SEAL

## DESCRIPTION

Chest Seal is an advanced occlusive dressing designed to treat open chest wounds. Penetrating injuries to the chest that result in open chest wounds may lead to tension pneumothorax, the second leading cause of preventable death on the battlefield. Chest Seals are specifically designed with a medical grade hydrogel adhesive strong enough to adhere to skin that is exposed to blood, sweat, hair, sand or water.

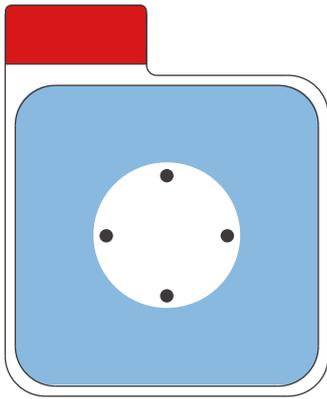


## FEATURE

1. High viscosity. 90 ° peel force test, the force value can be stable >70N/100mm.
2. The viscosity decays slowly. After accelerated aging for 5 months (equivalent to more than 5 years of real-time aging), the peel strength test can still be greater than 60N/100mm. Attenuation less than 10%.
3. Diverse and complete specifications and models.


## INSTRUCTIONS FOR USE




1 Clean and dry wound.

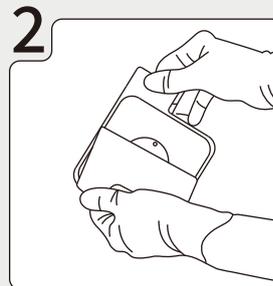

2 Remove protective liner from adhesive coated surface.

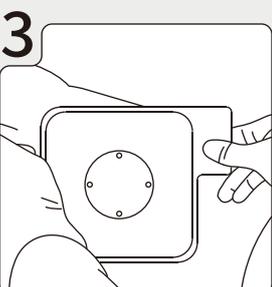


3 Place it on patient's wound, adhesive side down.

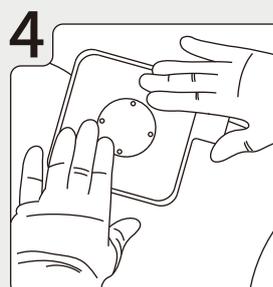

4 Press it firmly to skin to assure an occlusive seal.

# SPECIFICATION

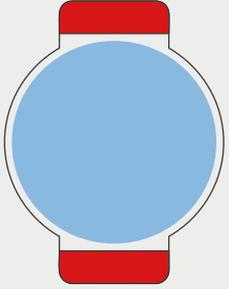


#PZXF0039




#PZXF0038



#PZXF0037

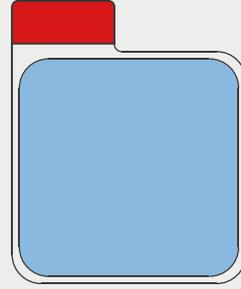

#PZXF0041

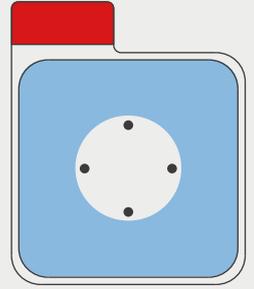

#PZXF0040

**MATERIAL: HYDROGEL**

## SIZE

155×106mm

155×106mm

130×106mm

208×160mm

208×160mm

## WEIGHT

40g

40g

40g

40g

40g



**EU REP** SUNGO Cert GmbH,  
Harffstr.47,40591 Düsseldorf, Germany  
E-mail: de.rep@sungoglobal.com



**EMSRUN GROUP**  
Room 101,201,301,No.9 Building,Huaqing Creative  
Park,Huishan Area,Wuxi,214000 Jiangsu,P.R.China  
E-mail: info@rhinorescuestore.com



**WUXI EMSRUN TECHNOLOGY CO., LTD.**  
9-201, Huaqing Chuangzhi Park, Huishan Economic  
Development Zone, Wuxi, Jiangsu Province,CHINA  
E-mail: info@emsrun.com

[www.rhinorescuestore.com](http://www.rhinorescuestore.com) | [info@rhinorescuestore.com](mailto:info@rhinorescuestore.com)